Handleidingen

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- [Accu-Chek Safe-T-Pro Plus](#)
- [Paracetamol 1g](#)

00009580 – Infucose – 120x60x110 (120x60) mm
 POF-Certified Foil Package Unit, 0.01

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- (18) Sterile, single-use lancing device for the fingertip or the palm of a child under 1 year, from the heel.**

Components of the single-use lancing device and illustration

- Description**

 - Housing
 - Penetration depth-adjuster
 - Starting cap of the needle
 - Penetration depth indicator
 - Low penetration depth (approx. 1.2 mm)
 - Medium penetration depth (approx. 1.8 mm, pre-set)
 - High penetration depth (approx. 2.3 mm)

Preparing to obtain blood

- To improve blood flow at 1 year and older, the capillary blood must be obtained from the heel. Obtaining blood must occur from the lateral areas of the sole on black areas in illustration 1.

The puncture must occur vertically in the sole area in the illustrations. While obtaining blood, the child should be held with its foot vertically to the floor. A toe which has already been punctured must not be punctured a second time.

To improve blood flow at the heel before obtaining blood, the heel should be warmed. Do not, however, wrap it in a hot towel, must only at a maximum 42 °C.

The heel must be disinfected and then dried completely. To obtain capillary blood from the heel of children under 1 year, use only the low or medium penetration depth.

1 year use only the low or medium penetration depth. If the high penetration depth is used, the calf veins of the heel are at risk of being injured.

Children of 1 year and adults
 We recommend obtaining capillary blood from the side of the foot or the heel after the region has been warmed on black areas in illustration 1. Any finger can be used for the puncture.

The patient's hands should be washed with warm water and soap, and then dried thoroughly. This ensures that the puncture site is clean and uncontaminated. If you are using alcohol swabs, the finger must also be completely dry.

Obtaining blood

- Removing the starting cap**

 - Twist the starting cap and remove it.
 - Do not use the single-use lancet device if the starting cap has been previously removed.
 - In extreme cases, the needle can stick out of the housing so that injury cannot be completely excluded. Discard the lancet single-use lancet device in accordance with your facility's disposal regulations.

Setting the penetration depth
 Turn the penetration depth adjuster to the desired penetration depth.

- Do not set the penetration depth below the remaining the starting cap.

 - In case capillary blood from the heel of children under 1 year, use only the low or medium penetration depth. If the high penetration depth is used, the calf veins of the heel are at risk of being injured.

- Obtaining blood**

 - Hold the single-use lancet device between the index finger and thumb and thumb.
 - Press the single-use lancet device firmly against the chosen puncture site.
 - Using your thumb, press the release button down until it presses on the foot to encourage a blood drop to form.
 - Gently squeeze the finger towards the fingertip or palm so that pressure on the foot to encourage a blood drop to form.
 - **For use as diagnostic tool, the finger must not be in the laboratory, removed or compressed, as this may produce incorrect test results.**
 - Always discard used single-use lancet device in accordance with your facility's disposal regulations.

Last update:
 2015-10

Accu-Chek Customer Care

Neche Diabetes Care Limited
 One West Avenue, Burgess Hill
 West Sussex, BN15 9RD United Kingdom
 Accu-Chek Customer Care (UK)
 UK Freephone number: 0800 701 020
 UK Freephone number: 1 800 709 600
 * Calls may be recorded for training purposes
 www.accu-chek.co.uk

Notes

- **Waste disposal**
 Accu-Chek Customer Care
 One West Avenue, Burgess Hill
 West Sussex, BN15 9RD United Kingdom
 Freephone: 0800 701 020

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© 2015 Roche Diabetes Care
ACCU-CHEK and SAFE-T-PRO are trademarks of Roche.