Accu-Chek Safe-T-Pro Plus

Eigen regels

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(EN) Sterile, single-use lancening device for obtaining capillary blood from the fingertip or, if the patient is a child under 1 year, from the heel.

Components of the single-use lancening device (see illustration)

Description

- ① Release button
- ② Housing
- ③ Penetration depth adjuster
- ④ Identity cap of the needle
- ⑤ Penetration depth indicator
- ⑥ Low penetration depth (approx. 1.3 mm)
- ⑦ Medium penetration depth (approx. 1.8 mm, pre-set)
- ⑧ High penetration depth (approx. 2.3 mm)

Warnings & Precautions

Children younger than 1 year
 With children younger than 1 year of age, the capillary blood must be obtained from the heel. Obtaining blood must not occur from the lateral areas of the sole (see black areas in illustration).

The puncture must occur vertically in the sole (see arrows in the enlargements). While obtaining blood, the child should be held with his foot vertically facing the floor. A site which has already been punctured must not be punctured a second time.

To improve blood flow at the heel before obtaining blood, you can carefully warm the heel, for example by wrapping it in a warm, moist cloth at a maximum 42 °C.

The heel must be dried and then dried completely.

To obtain capillary blood from the heel of children under 1 year, use **only the low or medium penetration depth**. If the high penetration depth is used, the calcaneus (heel bone) is at risk of being injured.

Children aged 1 year and adults
 We recommend obtaining capillary blood from the side of the fingertip, as this site requires the least amount of pain (see black areas in illustration). Any finger can be used for the puncture.

The patient's hands should be washed with warm water and soap, and then dried thoroughly. This ensures that the puncture site is clean and increases blood flow. If you are using alcohol wipes, the finger must also be completely dry.

Obtaining blood

Removing the sterility cap

- ▶ Twist the sterility cap and remove it.
- ▶ Do not use the single-use lancening device if the sterility cap has been previously removed.
- ▶ In extreme cases, the needle can stick out of the housing so that it may cannot be completely enclosed. Discard the defective single-use lancening device in accordance with your facility's disposal regulations.

Setting the penetration depth

- ▶ Turn the penetration depth adjuster to the desired penetration depth.
- ▶ Do not set the penetration depth before removing the sterility cap.
- ▶ To obtain capillary blood from the heel of children under 1 year, use **only the low or medium penetration depth**. If the high penetration depth is used, the calcaneus (heel bone) is at risk of being injured.

Obtaining blood

- ▶ Hold the lancening device between the index finger, middle finger and thumb.
- ▶ Press the single-use lancening device firmly against the chosen fingertip site.
- ▶ Using your thumb, press the release button down completely.
- ▶ Gently massage the finger towards the fingertip or exert slight pressure on the heel to encourage a blood drop to form.
- ▶ For some diagnostic tests, the fingertip/heel must not be vigorously massaged or compressed, as doing so may produce incorrect test results.
- ▶ Always discard used single-use lancening devices in accordance with your facility's disposal regulations.

ACCU-CHEK Customer Care

United Kingdom
 Roche Diabetes Care Limited
 Charles Avenue, Burgess Hill
 West Sussex, BN15 9PP, United Kingdom
 ACCU-CHEK Customer Care Line 1
 UK Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.co.uk

Deutschland

ACCU-CHEK Customer Care
 Roche Diabetes Care GmbH
 Grenzach-Wyhlen, 78165, Germany
 ACCU-CHEK Customer Care Line 1
 DE Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.de

France

ACCU-CHEK Customer Care
 Roche Diabetes Care SAS
 11, rue de l'Industrie
 92015 Nanterre Cedex, France
 ACCU-CHEK Customer Care Line 1
 FR Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.fr

Spain

ACCU-CHEK Customer Care
 Roche Diabetes Care S.L.
 Calle de la Industria, 11
 28911 Getafe, Madrid, Spain
 ACCU-CHEK Customer Care Line 1
 ES Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.es

Italy

ACCU-CHEK Customer Care
 Roche Diabetes Care S.p.A.
 Via dell'Industria, 11
 00144 Roma, Italy
 ACCU-CHEK Customer Care Line 1
 IT Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.it

Poland

ACCU-CHEK Customer Care
 Roche Diabetes Care Sp. z o.o.
 ul. Przemysłowa 11
 01-650 Warszawa, Poland
 ACCU-CHEK Customer Care Line 1
 PL Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.pl

China

ACCU-CHEK Customer Care
 Roche Diabetes Care (China) Co., Ltd.
 No. 11, Industrial Road
 Beijing 100144, China
 ACCU-CHEK Customer Care Line 1
 CN Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.cn

India

ACCU-CHEK Customer Care
 Roche Diabetes Care Pvt. Ltd.
 Plot No. 11, Industrial Area
 Phase II, Gurgaon, Haryana
 122002, India
 ACCU-CHEK Customer Care Line 1
 IN Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.in

Japan

ACCU-CHEK Customer Care
 Roche Diabetes Care (Japan) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 JP Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.jp

South Korea

ACCU-CHEK Customer Care
 Roche Diabetes Care (Korea) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 KR Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.kr

China (Taiwan)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Taiwan) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 TW Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.tw

China (Hong Kong)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Hong Kong) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 HK Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.hk

China (Singapore)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Singapore) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 SG Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.sg

China (Malaysia)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Malaysia) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 MY Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.my

China (Indonesia)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Indonesia) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 ID Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.id

China (Philippines)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Philippines) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 PH Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.ph

China (Vietnam)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Vietnam) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 VN Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.vn

China (Thailand)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Thailand) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 TH Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.th

China (Australia)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Australia) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 AU Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.au

China (New Zealand)

ACCU-CHEK Customer Care
 Roche Diabetes Care (New Zealand) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 NZ Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.nz

China (South Africa)

ACCU-CHEK Customer Care
 Roche Diabetes Care (South Africa) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 ZA Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.za

China (Brazil)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Brazil) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 BR Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.br

China (Mexico)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Mexico) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 MX Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.mx

China (Colombia)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Colombia) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 CO Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.co

China (Peru)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Peru) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 PE Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.pe

China (Chile)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Chile) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 CL Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.cl

China (Argentina)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Argentina) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 AR Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.ar

China (Ecuador)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Ecuador) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 EC Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.ec

China (Venezuela)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Venezuela) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 VE Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.ve

China (Cuba)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Cuba) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 CU Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.cu

China (Czech Republic)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Czech Republic) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 CZ Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.cz

China (Slovakia)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Slovakia) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 SK Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.sk

China (Slovenia)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Slovenia) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 SI Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.si

China (Croatia)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Croatia) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 HR Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.hr

China (Hungary)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Hungary) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 HU Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.hu

China (Poland)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Poland) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 PL Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.pl

China (Romania)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Romania) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 RO Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.ro

China (Bulgaria)

ACCU-CHEK Customer Care
 Roche Diabetes Care (Bulgaria) Co., Ltd.
 11-1, Industrial Road
 Chiyoda-ku, Tokyo 100-8585, Japan
 ACCU-CHEK Customer Care Line 1
 BG Freephone number: 0800 701 300
 RD Freephone number: 1 800 709 600
 *Calls may be recorded for training purposes
 www.accu-check.bg

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